

Acuity LTA-Mobile 8.10 Competency Test

Name (print): _____ Unit: _____ Date: _____

1. The Sort button allows you to sort rooms by one of the following:
 - a. Numerical order
 - b. Alphanumeric order
 - c. Last name first
 - d. First name first

2. True or False:
The icon in a patient's Waveform Window that allows you to reach the Arrhythmia Events Review window is the black heart icon (♥).

3. If a red latched alarm is indicated in a Waveform Window, it is informing you that which type of alarm has occurred:
 - a. Lethal or high-level arrhythmia alarm
 - b. Medium-level arrhythmia alarm
 - c. Low-level alarm
 - d. Alarm that is set to Off

4. If a red latched alarm is present in a Waveform Window, it remains until the following occurs:
 - a. You click on the red bar to clear it.
 - b. It is replaced by another latched alarm of equal or higher priority.
 - c. You click on the black heart icon to reach the Arrhythmia Events Review window.
 - d. All of the above

5. The Arrhythmia Events Review window can *only* be viewed in the:
 - a. Half screen review view
 - b. Full screen review view
 - c. Both
 - d. Neither

6. True or False:
Lethal alarms as defined by the Acuity System are V-Fib, Asystole, and V-Tach, and they have a specific high alarm tone that is different from all other alarm tones.

7. The Acuity System uses the following leads to analyze and label arrhythmias for Propaq LT or Micropaq monitors with 5-lead cables:
 - a. II, V, and III
 - b. V, I, and aVR
 - c. I, II, and III
 - d. V only

8. True or False:
The leads that the Acuity System uses to analyze and label arrhythmias for Propaq Encore or CS monitors are the same two leads that are displayed in the Virtual Monitor.

9. True or False:
The Arrhythmia Events Review list displays the arrhythmia type, alarm priority, day, time and duration of the event.

10. To post a patient note in a Waveform Window, such as “Patient is in X-Ray”, click on:
 - a. Review, Arrhythmia Events, Print
 - b. The Virtual Monitor icon, Setup, Patient ID, and then type in the Patient Note field.
 - c. The white area of the Waveform Window, and then type the note
 - d. Virtual Monitor icon, Review, Patient ID, and then type in the Patient Note field

Acuity LTA-Mobile 8.10 Competency Test Answer Sheet

Name: _____

Unit: _____

Date: _____

1. _____

2. _____

3. _____

4. _____

5. _____

6. _____

7. _____

8. _____

9. _____

10. _____

Acuity LTA-Mobile 8.10 Competency Test Answer Key

1. b
2. True
3. a
4. d
5. c
6. True
7. a
8. True
9. True
10. b