

## Acuity® Workstation 8.10 and Arrhythmia Competency Checklist

Name (print): \_\_\_\_\_ Unit: \_\_\_\_\_ Date: \_\_\_\_\_

- \_\_\_\_\_ True or False: When editing a patient's information in the Patient ID window, you can also change the patient's ID number.
  
- \_\_\_\_\_ Locate the Arrhythmia Alarms Setup Window. What three leads are used to analyze and label arrhythmias?
  
- \_\_\_\_\_ Discuss the Single ECG feature under the Arrhythmia Alarm Setup Window. When is the only time this should be utilized?
  
- \_\_\_\_\_ Will Arrhythmia Detection label an arrhythmia if you select OFF as an alarm level? What is the difference between HIGH, MEDIUM, LOW and OFF alarms?
  
- \_\_\_\_\_ Locate the Relearn button and discuss its purpose.
  
- \_\_\_\_\_ Review the Arrhythmia Event History directly from the patient waveform window. Now move to the next or previous arrhythmia event.
  
- \_\_\_\_\_ Locate the Tabular Trends/Events Review window. Select a 15-minute Tabular review and print.
  
- \_\_\_\_\_ If a red bar or half red/half yellow bar appears in the Waveform Window, what does that mean?
  
- \_\_\_\_\_ Open the Patient List window and demonstrate how to properly discharge a patient from Acuity; review the stored data for that patient