

# Connex® Vital Signs Monitor 6000 Series with Connex® VM Competency Checklist – Wireless Workflow

Name (print): \_\_\_\_\_ Unit: \_\_\_\_\_ Date: \_\_\_\_\_

- \_\_\_\_\_ Locate the power button and turn on the device.
- \_\_\_\_\_ Demonstrate how to ID the Clinician.
- \_\_\_\_\_ Scan the patient's barcode. Discuss the importance of visually confirming the patient's information.
- \_\_\_\_\_ Demonstrate how to enter a Respiration Rate or Pain Level.
- \_\_\_\_\_ Touch **Patients** tab; now touch **Modifiers** and enter information for patient position and BP site.
- \_\_\_\_\_ Touch **Save** to send the patient data to Connex® VM.

*(if applicable) To send readings taken during intervals:*

- \_\_\_\_\_ Locate the **Review** tab; touch the readings to send, then touch **Send** and then **OK** to confirm send.
- \_\_\_\_\_ True or False. Depending upon configurations, once you touch **Send**, data is sent to Connex® VM and the memory on the device is cleared.