








# The ABC's of Micropaq

<p><b>A</b></p> <p>Age Ranges</p>	<p>Intended for use on Pediatric and Adult patients only. Pediatric carrying pouch only intended for ambulatory pediatric patients weighing 40-80lbs.</p>
<p><b>B</b></p> <p>Battery Info</p>	<p>Lithium Ion battery Duration:</p> <ul style="list-style-type: none"> <li>• ECG only: 25hrs</li> <li>• ECG &amp; SpO2: 10hrs</li> </ul> <p>Store battery in charger when not in use. Conditions based on:</p> <ul style="list-style-type: none"> <li>• fully charged battery operating at 25° C, connected to Acuity</li> <li>• minimal artifact</li> <li>• 8 patient alarms per hour</li> </ul> <p>Battery reducing factors:</p> <ul style="list-style-type: none"> <li>• not connected to Acuity</li> <li>• alarms and alerts long in duration</li> <li>• operator key usage (active display)</li> <li>• battery age</li> <li>• motion artifact during SpO2 monitoring</li> </ul>
<p><b>C</b></p> <p>Cleaning</p>	<p><b>Monitor, Battery, &amp; Charger</b> Wipe with a nearly-dry cloth moistened with an approved cleaning solution:</p> <ul style="list-style-type: none"> <li>• Warm water, liquid soap, Coverage®, Windex®, Ovation®, Hydrogen peroxide, Wex-cide®©, T.B.Q.®©</li> </ul> <p>After cleaning, remove excess solution with a soft cloth dampened with clean water, then dry. Do not let solution get into connector openings or crevices. Do not autoclave or immerse.</p> <p><b>Accessories</b> Consult manufacturer's instructions.</p>
<p><b>D</b></p> <p>Defaults</p>	<p>When the monitor connects to Acuity for a new patient, the Acuity Central Station downloads the appropriate default setting stored at Acuity to the device.</p>
<p><b>E</b></p> <p>End Monitoring</p>	<p><b>Standby/Temporary Disconnect</b> Remove cables from the top of the device. The monitor will display an <b>EQUIPMENT ALERT</b> message. Press the  key to acknowledge the alert and the device will remain in <b>STANDBY</b> (<b>BLUE</b> state at Acuity) until the cables are replaced.</p> <p><b>End Tele/Permanent Disconnect</b> To discontinue use of the monitor on a patient, use the following steps:</p> <ol style="list-style-type: none"> <li>1. Press the <b>DOWN</b>  button to display the <b>MAIN MENU</b> (if menu lockout is enabled, press the <b>SELECT</b>  button at the same time).</li> <li>2. Continue to press the  button to highlight <b>ACUITY</b> and press the  button.</li> <li>3. Press  button to highlight <b>END TELE</b>, then press .</li> <li>4. When the monitor displays <b>SAFE TO REMOVE BATTERY</b>, remove the battery to power down the monitor.</li> </ol>