



Micropaq Competency Test


Name (print): _____ Unit: _____ Date: _____



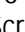


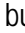


1. How do you turn the Micropaq on?
 - a. Attach a cable to top of monitor
 - b. Attach a battery to the back of the monitor
 - c. Press the On/Off button
 - d. None of the above

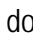


2. To suspend a patient alarm or equipment alert you would:
 - a. press the Select  button
 - b. remove the battery
 - c. press the Snapshot  button
 - d. scroll up and down until you find the suspend button



3. Which of the following should **not** be used to clean the Micropaq?
 - a. Warm water
 - a. Liquid soap
 - b. Alcohol products
 - c. Cidex

4. True or False:
When a Micropaq is out of range, it will act as a standalone monitor and will display patient data as well as generate patient alarms.

5. True or False:
If the Micropaq is properly connected to the Acuity workstation, a green light in the upper right corner and a wireless symbol  will appear on the screen.

6. To change alarm limits you would:
 - a. press Scroll Down  to highlight LIMITS, then press Select  to display Alarm Limits Adjust Menu, then press Scroll Down  or Scroll Up  to highlight the desired limit, then press Select  to display the Threshold Adjustment Menu. Scroll and select the + or – selections to change the limit as desired.
 - b. press the Select  button until you change alarm values
 - c. press the Nurse's Cap  button, select alarm value to be changed and then press the Select  button
 - d. none of the above

7. To **discontinue** monitoring a patient at the Micropaq:
 - a. scroll down to highlight Acuity, press the Select  button, then press the Scroll Down  button to highlight END TELE, then press Select . When the Micropaq displays the message SAFE TO REMOVE BATTERY, remove the battery. Disconnect leads and sensors from the patient.
 - b. disconnect the leads from the patient and then press Snapshot button
 - c. discharge patient from Acuity and then disconnect patient from leads
 - d. both a and b

8. True or False:
If the Menu Lockout is configured to On, in order to gain access to the Main Menu you must simultaneously press and release the Select  and Scroll Down  buttons.

9. True or False:
In order to extend battery life, the display on the Micropaq becomes blank after two minutes if no button is pressed. The display reappears if an alarm or alert occurs, any button is pressed, or if a cable is inserted.

10. If an alarm or alert occurs, where will it be displayed?
 - a. At the Acuity Workstation
 - b. On the Micropaq itself
 - c. On the optional hallway message panel(s)
 - d. All of the above

Micropaq Competency Test

Answer Sheet

Name: _____

Unit: _____

Date: _____

1. _____

2. _____

3. _____

4. _____

5. _____

6. _____

7. _____

8. _____

9. _____

10. _____

Micropaq Competency Test

Answer Key

1. B
2. A
3. C
4. True
5. True
6. A
7. A
8. True
9. True
10. D