

Connex® ProBP™ 3400 Digital Blood Pressure Device Competency Checklist

Name (print): _____ Unit: _____ Date: _____

- ___ Locate the Power-On button and turn on the ProBP 3400.
- ___ Demonstrate how to properly charge the ProBP 3400.
- ___ True or False: The battery for the ProBP is fully charged if the battery LED displays a green indicator light.
- ___ True or False: The ProBP 3400 device is intended to take systolic and diastolic blood pressure readings on Adult, Pediatric and Neonate patients.
- ___ True or False: If a patient falls into the range between a regular and large blood pressure cuff, you should use the larger cuff.
- ___ Start a Blood Pressure Measurement with the ProBP 3400; then abort a blood pressure in progress.
- ___ (If Applicable) True or False: The Sure BP Blood Pressure Algorithm takes a BP upon inflation only and will never step deflate.
- ___ Locate the Review menu to recall BP readings.
- ___ True or False: The ProBP 3400 device retains up to 50 readings in memory.
- ___ True or False: Cavi-Wipes™, Sani-Cloth® Plus, 10% chlorine bleach solution, and 70% alcohol are all approved cleaning solutions for the device.