











The ABC's of Spot LXi Vital Signs Device

<p>A</p> <p>Age Ranges</p>	<p><u>NIBP</u> not for use on neonates (less than 28 days of age or 44 weeks gestational age).</p> <p><u>Thermometry</u></p> <p> Oral Mode: all ages; use probe with BLUE ejection button and probe well</p> <p> Adult Axillary Mode: patients age 18 years and older; use probe with BLUE ejection button and probe well</p> <p> Pediatric Axillary Mode: patients age 17 years and younger; use probe with BLUE ejection button and probe well</p> <p> Rectal Mode: all ages; use probe with RED ejection button and probe well</p> <p>Tympanic thermometer (if applicable): appropriate for use on all ages</p>
<p>B</p> <p>Battery Info</p>	<p>Uses a sealed lead-acid battery. Battery must be connected upon initial receipt per instructions on battery door. Charge for 6 hours (+1 hour with Braun 4000 device) prior to initial use OR charging icon on screen ceases to flash. Charge for 4 additional hours if device has a printer. Charge when not in use. Duration of battery is model specific:</p> <ul style="list-style-type: none"> • 45MTO = 14.5 hours • 45NTO = 13 hours • 45MEO = 13.5 hours • 45NEO = 12.5 hours
<p>C</p> <p>Cleaning</p>	<p><u>Device</u> Wipe the device with a cloth slightly dampened with appropriately diluted, non-staining disinfectant solution. Use either 70% isopropyl alcohol, 10% chlorine bleach solution, mild detergent in warm water, or PDI Sani-System cloths. Never immerse. Prevent water or fluids from entering any connectors. Do not sterilize or autoclave the Spot LXi device.</p> <p><u>Accessories</u> Do not immerse, soak or autoclave. Clean removable probe well by swabbing with a cloth dampened with 70% isopropyl alcohol or 10% bleach solution. Probe well can be immersed. Ensure probe well is dry before re-inserting into device.</p>
<p>D</p> <p>Defaults</p>	<p>To enter the Internal Configuration menu, follow these steps starting with the device powered OFF:</p> <ol style="list-style-type: none"> 1. Press and hold the POWER  and SELECT  buttons simultaneously. The Internal Configuration menu will display on the screen. 2. Use the NAVIGATION  buttons to move through the menu options. 3. Use the SELECT  button to access options or accept a change. 4. Press the POWER  button to exit the Internal Configuration menu.
<p>E</p> <p>End Use</p>	<p>Press the POWER  button to turn device off. After 2 minutes of inactivity, the device goes into STANDBY mode. The screen will go blank; press any key to activate device. After 30 minutes of inactivity, the device powers itself off.</p>