

# Spot Vital Signs® LXi Competency Checklist

Name: \_\_\_\_\_ Unit: \_\_\_\_\_ Date: \_\_\_\_\_

- \_\_\_\_\_ Locate the power button and turn on the Spot Vital Signs® LXi
- \_\_\_\_\_ True or False. The Spot Vital Signs® LXi can be used to obtain systolic and diastolic blood pressure readings on patients greater than 29 days old.
- \_\_\_\_\_ Obtain a manual BP from the Spot Vital Signs® LXi; then cancel a blood pressure in progress
- \_\_\_\_\_ True or False. The Spot Vital Signs® LXi only measures BP during cuff inflation.
- \_\_\_\_\_ (if applicable) Obtain an oral temperature. How will probe location impact the results?
- \_\_\_\_\_ (if applicable) Explain how to adjust thermometer settings from Adult Axillary to Pediatric Axillary
- \_\_\_\_\_ (if applicable) Demonstrate proper placement of the probe into the ear canal and take an ear temperature
- \_\_\_\_\_ (if applicable) True or False. When obtaining SpO2 readings, you **must** remove the sensor in order to store the result in memory.
- \_\_\_\_\_ Locate the Memory button on the Spot® LXi; how many readings can be recalled?
- \_\_\_\_\_ Discuss how to enter Height, Weight, Respirations and Pain Level into the Spot Vital Signs® LXi