






The ABC's of Spot Vital Signs Device

<p>A</p> <p>Age Ranges</p>	<p>Not intended for use on neonates (28 days of age and less OR less than 44 gestational weeks). No mode changes required for use on Pediatric or Adult patients. Use appropriate cuff size for NIBP.</p> <p>Thermometry Oral (OrL) Mode: all ages; use probe with BLUE ejection button. Axillary (ALY) Mode: use on patients under the age of 4 years old. If an axillary temperature is desired on patients older than 4 years, use the thermometer in OrL mode. Rectal (REC) Mode: all ages; use thermometer probe with RED ejection button.</p>
<p>B</p> <p>Battery Info</p>	<ul style="list-style-type: none"> • Uses Welch Allyn 4200-84 lead acid battery. • Prior to initial use, charge for 16 hours. • When fully charged, device supports approximately 130 blood pressure cycles at 7-minute intervals. • Keep plugged in when not in use. • Takes approximately 12 hours to full charge battery.
<p>C</p> <p>Cleaning</p>	<p>Device Wipe the device with a cloth slightly dampened with appropriately diluted, non-staining disinfectant solution. Use either 70% isopropyl alcohol, 10% chlorine bleach solution, or mild detergent in warm water. Never immerse. Prevent water or fluids from entering any connectors. Do not sterilize or autoclave. Do not use ethyl alcohol to clean Spot Vital Signs device.</p> <p>Accessories Do not immerse, soak, or autoclave. Clean removable probe well by swabbing with a cloth dampened with 70% isopropyl alcohol or 10% bleach solution. Probe well can be immersed. Ensure that probe well is dry prior to re-insertion in device.</p>
<p>D</p> <p>Defaults</p>	<p>To change default configurations, follow these steps starting with the device OFF.</p> <ol style="list-style-type: none"> 1. Simultaneously press and hold the POWER  and BP Start/Stop  buttons. The device will enter the internal configuration mode. 2. Press the MODE  button to cycle through the Internal Configuration menu until you see the menu option displayed on the screen. 3. Use the Next Patient/Clear/Cancel  button or the Blood Pressure Start/Stop button to change the default setting. 4. Press the MODE button once to save the change and press the POWER button to exit the Internal Configuration Mode.
<p>E</p> <p>End Use</p>	<p>To turn device off, press the POWER  button. After 2 minutes of inactivity, the device goes into Standby mode. "ZZZ" will appear on the screen. Press the MODE button to activate device.</p>